

MDPS100

## SELF-ADHESIVE GEL PADS USER GUIDE



**ChoiceMMed™**

### Introduction

---

*The MDPS100 ChoiceMMed Self Adhesive Replacement Gel Pads are only for use with ChoiceMMed Electronic Pulse Stimulators.*

Box includes:  
10 - 1.57"x 1.57" Replacement  
Gel Pads with Electrode Connectors.  
User Guide.

Pads consist of gel, carbon film, cloth backing, and the electrode connector is biocompatible.

The electrode pads are reusable up to 80 uses per pad. They should be replaced when their adhesiveness becomes weak.

To insure proper use, ChoiceMMed Self Adhesive Replacement Gel Pads MDPS100 should only be used with ChoiceMMed Electronic Pulse Stimulators.

### How to Use


---

Insert the pins of the connecting wire into the wire connectors of each electrode pad (as shown in image A).

Remove the electrode pads from the protective liner. Lift at the edge of the electrode pad and peel from the liner. Do not pull on the lead wires when removing the electrode pads from the protective liner or skin. The electrodes may become damaged (shown in image B).

Apply firmly to the skin. \*Clean skin thoroughly prior to each application of electrode pads. Pads will not stick well if any lotion, make-up, or dirt is left on your skin. Make sure the Electronic Pulse Stimulator is off before applying the electrodes to the skin.

When not in use, return the pads to the protective liner (as shown in image C). Store the electrode pads in the re-sealable bag in a cool place, out of direct sunlight. Reusable up to 80 uses.

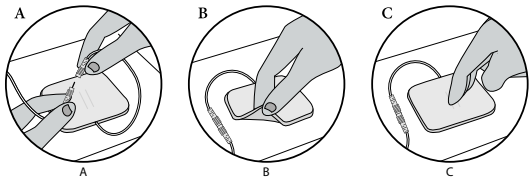

### Positions For Use



SHOULDER/NECK



SOLES OF FEET



WAIST/BACK



HANDS/ARMS



LEGS/JOINTS

### WARNINGS (READ BEFORE USE)

**Do not** let the sticky side of the electrode pads touch anything, including your finger tips.

**Do not** use damaged or broken wires.

**Do not** attempt to move the electrode pads while the device is operating.

**Do not** use the Electronic Pulse Stimulator device if you have an implanted defibrillator or implanted metallic devices, such as a pacemaker. Do not use on children or over open wounds or damaged skin.

**Do not** use the Electronic Pulse Stimulator device around the heart, on the head, mouth, pudendum or blemished skin areas.

**Stop** using the Electronic Pulse Stimulator device at once if you feel pain, discomfort, dizziness or nausea and consult your physician.

**\*\*Refer to your Electronic Pulse Stimulator manual for a full list of safety warnings before operating this unit.**